


Instructions for use


The Hegenberger Retractor is intended for women after a vaginal delivery. Only to be used by a clinical educated professional after understanding the instructions for use.




1 Open the pouch and remove the retractor. Hold the retractor as shown in the image.

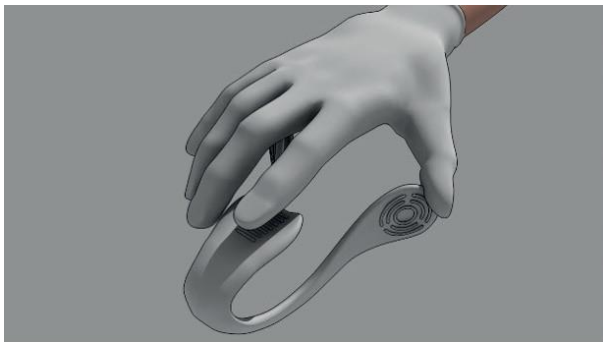

2 Gently press the sides of the retractor together before inserting.

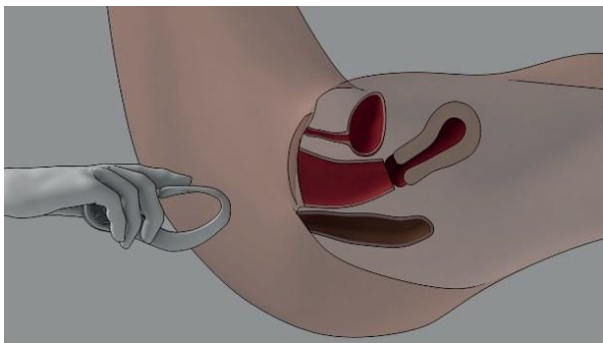


3 It is important that the patient is in a supine position with the legs elevated in stirrups and a well-functioning local anaesthetic.

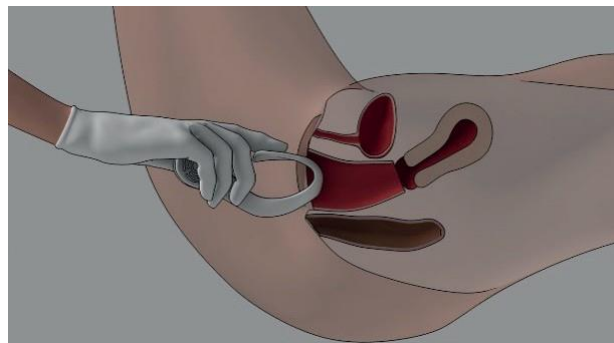


4 Keep the sides of the retractor compressed while inserting it in the vagina. **Do not apply lubrication.**



Instructions for use

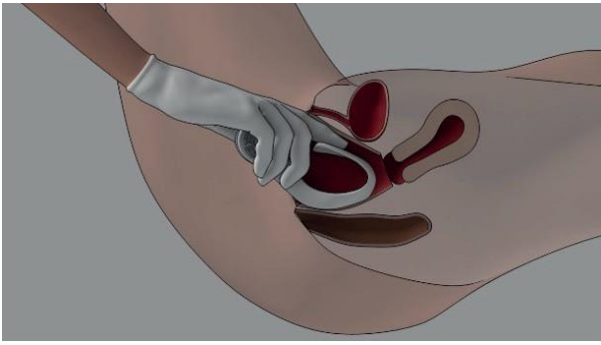


- 5 Ensure that the top of the retractor has been inserted past the pubic bone before releasing the sides.

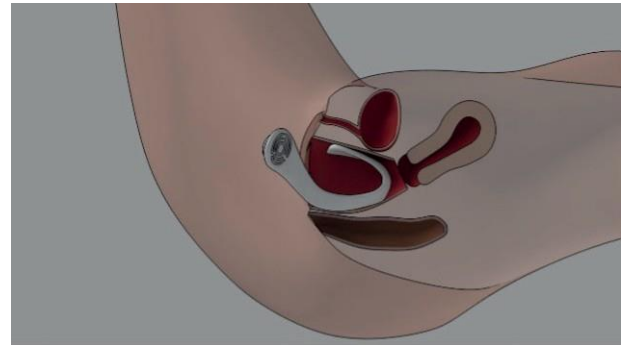

- 6 Correct placement is shown in this image.

Removing the Retractor

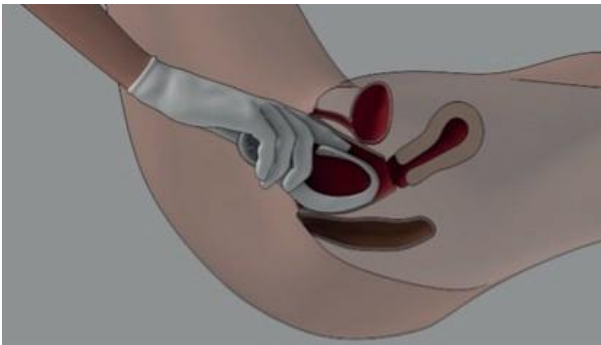

- 7 First you gently push the retractor a little further in to disengage it from behind the pubic bone. Then you place your index finger on the top piece to ensure the retractor is clear of the cervix. Gently press the wings of the retractor together, press down on the top part and gently retract the retractor from the Vagina.

Contraindications

The retractor is only intended for use on women following vaginal delivery. It is only intended for use by trained clinical professionals after understanding the instructions for use. Only to be used in clinical settings or hospitals. The patient must be in a supine position with elevated legs. Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion. Not to be used in cases involving post-partum haemorrhage. Not to be used during pregnancy. Not to be used for anterior wall procedures. Not for reuse.