BRUKSANVISNING

Hegenberger Retractor

A1390

Bruksanvisning.....	2
Instruction for use.....	3
Brugsanvisning.....	4
Brugsanvisning.....	5
Käyttöohjeet.....	6
Gebrauchsanweisung.....	7
Instructions d'utilisation.....	8
Gebruiksaanwijzing.....	9
Instrukcja użytkowania.....	10
Kasutusõpetus.....	11
Naudojimo instrukcijos.....	12
Lietošanas instrukcija.....	13
Istruzioni per l'uso.....	14
Οδηγίες χρήσης.....	15
instruções de uso.....	16
Instrucciones para el uso.....	17
Uputstvo za upotrebu.....	18
Návod k použití.....	19

sv

en

no

da

fi

de

fr

nl

pl

et

lv

lt

it

el

pt

es

sr

cs

Wing Plast AB

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Instructions For Use, Hegenerber Retractor


en


1. Open the pouch and remove the Hegenerber Retractor. Hold the Hegenerber Retractor as shown in the image.

2. Use your index finger as shown in the picture and press the sides of Hegenerber Retractor together before inserting. **Do not use gel or lubrication on Hegenerber Retractor.**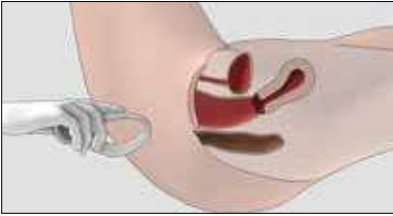


3. The patient must be in a supine position with the legs elevated.

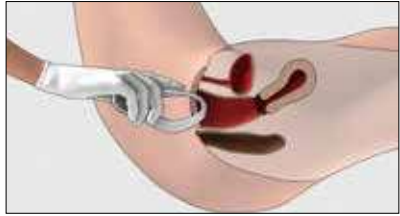
4. Keep the sides of Hegenerber Retractor compressed while inserting it in the vagina. Follow the direction of the pelvic canal.

5. Place Hegenerber Retractor all the way in so that the circles are resting on labia majora.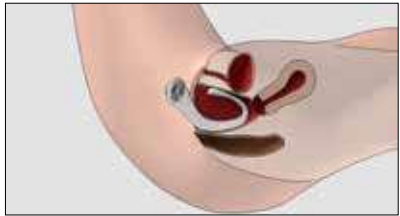

6. Correct placement of Hegenerber Retractor is shown in this image.

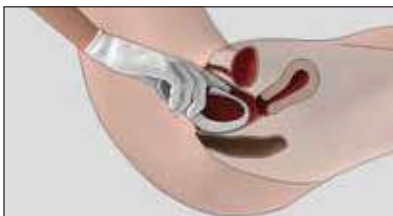Removing Hegenerber Retractor

7. Gently press the sides of Hegenerber Retractor together before removing.

It is important that you use your index finger to ensure the Hegenerber Retractor is clear of the cervix.

Press down on the top part and gently retract Hegenerber Retractor from the vagina.

Contraindications

Hegenberger Retractor is only intended for use on women following vaginal delivery.

Hegenberger Retractor is only intended for use by trained clinical professionals after understanding the instructions for use.

The patient must be in a supine position with elevated legs.

Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion.

Not to be used in cases involving post-partum haemorrhage.

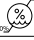
Not to be used without informed consent from the patient.

Not to be used during pregnancy.

Not to be used for anterior wall procedures.

Not for reuse.

Explanation of labeling on labels and instructions for use.

 See Instructions for use	 Manufacturing day	 Don't reuse
REF Article number	 Expiration date	 Don't sterilize for reuse
LOT Lot number	 Don't use if package is damaged	 Temp range 0 to 30 °C
MD Medical Device	 Humidity 0-70%	STERILE Sterilized with ethylene oxide
 Keep dry	 Protect from sunlight	 Latex free
 In accordance with the Medical Device Directive 93/42 / EEC (0402).		 Manufacturer

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Wing Plast AB, Nitgatan 11, 333 33 Smålandsstenar, Sweden, +46 371 330 30, www.wingplast.se